

## DISPENSING INTO DISPOSABLE DISPENSING DISH (BOTTLE)



Bottle cover "1" and "2" colors correspond to product bottle caps and labels. Please ensure bottle covers are applied correctly prior to use.

Bottle covers help insulate G2-BOND Universal from excess heat and give greater control of dispensing drops to minimize wastage. Their use is highly recommended. Bottle covers may be reused with subsequent refills. Clean with water or wipe with alcohol. Do not sterilize, autoclave, or clean with chemical disinfectants.

Shake the bottle and dispense 1-PRIMER into "1" in disposable dispensing dish.

Dispense 2-BOND into "2" in disposable dispensing dish.

## ETCHING MODE SELECTION



Etch for 10-15 sec., rinse for 5 sec. and dry.

## Light curing time

Light curing unit	Distance from light guide tip	
	<10mm	>10mm
Halogen / LED (700-1200 mW/cm <sup>2</sup> )	10 sec.	20 sec.
High power LED (more than 1200 mW/cm <sup>2</sup> )	5 sec.	10 sec.

## BONDING PROCEDURE (INDICATIONS FOR USE [1])



## INTRAORAL REPAIR (INDICATIONS FOR USE [3] [4])



## SEALING OF TOOTH PREPARATION (INDICATIONS FOR USE [6])

