

Calibra® VENEER

ESTHETIC RESIN CEMENT

TECHNIQUE GUIDE



1 APPLY CALIBRA® TRY-IN PASTE

Try-In Paste is a guide for cement shade range only.



2 CLEAN AND DRY

Thoroughly clean all internal surfaces of the veneer with water spray and dry.



3 APPLY CALIBRA SILANE COUPLING AGENT

Apply Calibra Silane to the veneer as outlined in the manufacturer's Direction for Use. Air dry the silanated surface of the restoration with clean, dry air from a dental syringe or allow to air dry.




4 APPLY ADHESIVE TO VENEER

Apply a single coat of Prime&Bond® NT™ adhesive to the internal surface of the veneer.*



5 AIR DRY

Air dry for 5 seconds.*

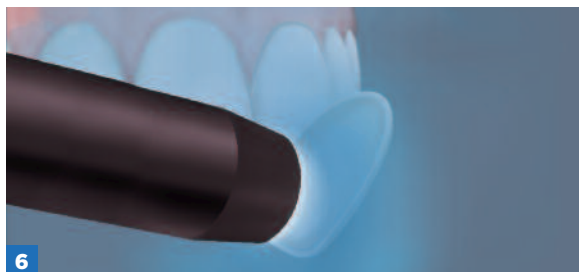

6 LIGHT CURE

Light cure adhesive for 10 seconds.*



7 APPLY TOOTH CONDITIONER GEL*

Apply Caulk® 34% Tooth Conditioner Gel to available enamel (and dentin if desired) for 15 seconds. Rinse for 10 seconds. Blot dry to keep moist. Do not rub.
(In Europe: DeTrey Conditioner 36%)



8 APPLY ADHESIVE TO TOOTH

Apply Prime&Bond NT adhesive to thoroughly wet tooth surface. This surface should remain wet for 20 seconds, which may necessitate additional adhesive.*



9
AIR DRY
 Gently air dry for 5 seconds.*



10
LIGHT CURE
 Light cure adhesive for 10 seconds*



11
APPLY CALIBRA® VENEER CEMENT
 Dispense the desired Calibra Veneer Cement shade directly onto the veneer. Protect cement from exposure to light.




12
SEAT
 Seat restoration.



13
CLEAN UP EXCESS
 Tack the veneer in place by briefly light curing the gingival portion only for no more than 10 seconds.



14
REMOVE EXCESS CEMENT
 Remove mylar strip and lift off excess cement around margins.

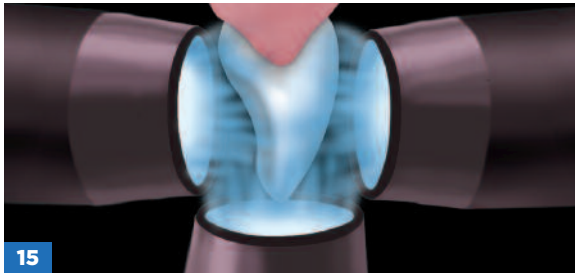

15
LIGHT CURE
 Light cure all marginal areas for 20 seconds from each direction - buccal, lingual and interproximal aspects.



16
FINISH
 Removal of resin cement flash and finishing of the margins is best accomplished with the Enhance® Finishing System (see complete Directions for Use).

Rx Only **Caution, Consult**
 For Dental Use Only Instructions for Use



See webpage for Instructions for Use
www.caulk.com • www.dentsply.eu/ifu

1-800-532-2855 / 1-302-422-4511
 1-800-263-1437 Canada

*Or follow manufacturer's Directions for Use.
 ©2016 Dentsply Sirona Inc. All rights reserved. **507102** (R 9/14/16)